

Triathlon® TS Offset Adapter Trial Modified

Protocol Addendum

Trial Assembly

Tibia

If an offset was prepared for, thread the appropriate size Stem Trial into the appropriate Offset Adapter Trial.

Time the rotation of the Offset Adapter Trial to the position recorded from the Tibial Offset Bushing. Align the scribe line on the Offset Adapter Trial to the scribe line on the Tibial Baseplate boss and snap the Offset Adapter Trial into the Tibial Baseplate Trial.

To disassemble the offset adapter trial, insert the key on the Universal Counter Wrench into one of the three slots in the Offset Adapter Trial as shown.



Femur

If an offset was prepared for, thread the appropriate size Stem Trial into the appropriate size Offset Adapter Trial.

Time the rotation of the Offset Trial to the position recorded from the Femoral Offset Bushing. Align the scribe line on the Offset Adapter Trial to the scribe line on the femoral boss and snap the Offset Adapter Trial into the Femoral Trial.

To disassemble the offset adapter trial, insert the key on the Universal Counter Wrench into one of the three slots in the Offset Adapter Trial as shown.

Joint Replacements

Trauma, Extremities & Deformities

Craniomaxillofacial

Spine

Biologics

Surgical Products

Neuro & ENT

Interventional Spine

Navigation

Endoscopy

Communications

Imaging

Patient Care & Handling Equipment

EMS Equipment

325 Corporate Drive
Mahwah, NJ 07430
t: 201 831 5000

www.stryker.com

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Triathlon. All other trademarks are trademarks of their respective owners or holders.

Literature Number: **LSPK48AD3**
MS/GS 6/10

Copyright © 2010 Stryker
Printed in USA