

Distal Femoral Augment Trial

Surgical Protocol Addendum

Distal Femoral Augment Trial Assembly

If distal augments are required in a Triathlon TS setting, assemble the Distal Femoral Augment Trial(s) to the surface(s) of the same size Femoral Trial component corresponding to augmentation resections.

Note: Distal Femoral Augment Trials are size specific and offered in 5mm, 10mm, and 15mm thicknesses. See table.

Assemble the Distal Femoral Augment Trial from the side as shown in Figure 1. Ensure that both Distal Femoral Augment Trial tabs have engaged the undercuts of the Femoral Trial as shown in Figure 2. Incorrect assembly is depicted in Figures 3 and 4 where the tabs fail to engage the undercuts.



Figure 1

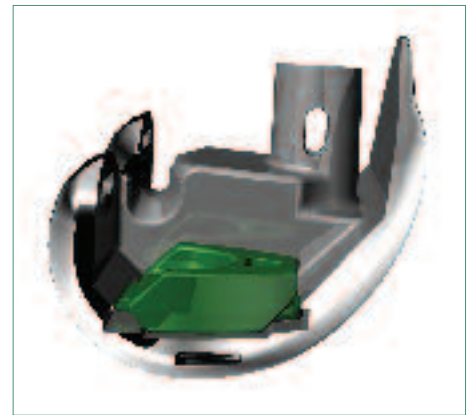


Figure 2



Figure 3



Figure 4

Distal Femoral Augment Trial

Surgical Protocol Addendum

Catalog No.	Description
Distal Femoral Augment Trial Surgical Protocol Addendum	
5540-T-100A	Triathlon Femoral Distal Augment Trial, 5mm – #1
5540-T-200A	Triathlon Femoral Distal Augment Trial, 5mm – #2
5540-T-300A	Triathlon Femoral Distal Augment Trial, 5mm – #3
5540-T-400A	Triathlon Femoral Distal Augment Trial, 5mm – #4
5540-T-500A	Triathlon Femoral Distal Augment Trial, 5mm – #5
5540-T-600A	Triathlon Femoral Distal Augment Trial, 5mm – #6
5540-T-700A	Triathlon Femoral Distal Augment Trial, 5mm – #7
5540-T-800A	Triathlon Femoral Distal Augment Trial, 5mm – #8
5541-T-100A	Triathlon Femoral Distal Augment Trial, 10mm – #1
5541-T-200A	Triathlon Femoral Distal Augment Trial, 10mm – #2
5541-T-300A	Triathlon Femoral Distal Augment Trial, 10mm – #3
5541-T-400A	Triathlon Femoral Distal Augment Trial, 10mm – #4
5541-T-500A	Triathlon Femoral Distal Augment Trial, 10mm – #5
5541-T-600A	Triathlon Femoral Distal Augment Trial, 10mm – #6
5541-T-700A	Triathlon Femoral Distal Augment Trial, 10mm – #7
5541-T-800A	Triathlon Femoral Distal Augment Trial, 10mm – #8
5542-T-100A	Triathlon Femoral Distal Augment Trial, 15mm – #1
5542-T-200A	Triathlon Femoral Distal Augment Trial, 15mm – #2
5542-T-300A	Triathlon Femoral Distal Augment Trial, 15mm – #3
5542-T-400A	Triathlon Femoral Distal Augment Trial, 15mm – #4
5542-T-500A	Triathlon Femoral Distal Augment Trial, 15mm – #5
5542-T-600A	Triathlon Femoral Distal Augment Trial, 15mm – #6
5542-T-700A	Triathlon Femoral Distal Augment Trial, 15mm – #7
5542-T-800A	Triathlon Femoral Distal Augment Trial, 15mm – #8



A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Triathlon. All other trademarks are trademarks of their respective owners or holders.

Literature Number: **LSPK48AD4**
MS/GS 12/10

Copyright © 2010 Stryker
Printed in USA